## A Step-by-Step Guide to the Application of ZIPZOC Zinc Oxide Medicated Stocking

- Assess the length and size of the limb to be dressed and cut the stocking to size, remembering that when the stocking is stretched laterally, it will shorten slightly. Alternatively, if the stocking is too long, the excess may be simply folded back on itself rather than cutting away.
- **2.** To apply, simply roll the stocking up onto the limb in a similar way to applying a sock. The stocking may be applied in direct contact with the skin or wound.
- **3.** Pull up towards the knee area and smooth away any wrinkles or folds before covering with a retention bandage, garment, or compression therapy system\*.
- **4.** To remove **ZIPZOC**, roll away down the limb, discard and re-apply a new sterile **ZIPZOC** stocking.

\*Compression therapy systems should only be applied on the advice of a clinician following thorough assessment.

## NOTE

**ZIPZOC** may be used in other parts of the body, e.g. on an upper limb to treat skin conditions. Cut the stocking to size, apply to the affected area and cover with a retention bandage or garment.







For further information on the application of **ZIPZOC** Medicated Stocking please visit our short product tutorial video at:





## pastebandagesevolan.com

ZIPZOC

ZIPZOC (zinc impregnated medicated stocking) Abbreviated prescribing information. Refer to summary of product characteristics before prescribing. Composition: A sterile rayon stocking impregnated with an ointment containing 20% Zinc Oxide Eachstocking(80Cmx Xrom) containsabout 41.5gofointment. Therapeuticindications: Aid to the treatment of chronic/leguleers and other associated skin conditions. Where chronic venous insufficiency exists, there medicated stocking came weekly. For topical application, ZIPZOC should be applied to cover the lower leg from the base of the toes to below the knee. All folds should be smoothed out: To protect tolking, a suitable outer bandage should be worn. If chronic venous insufficiency exists, There are application. ZIPZO Came and are compression bener application. ZIPZO Came and application, ZIPZO Came and application. ZIPZO Came and application. ZIPZO Came and application. ZIPZO Came and application and application. ZIPZO Came and app