


VORTEX AIR

The new platform for
macerator technology

Smarter diagnostics
Sustainable design
Superior performance

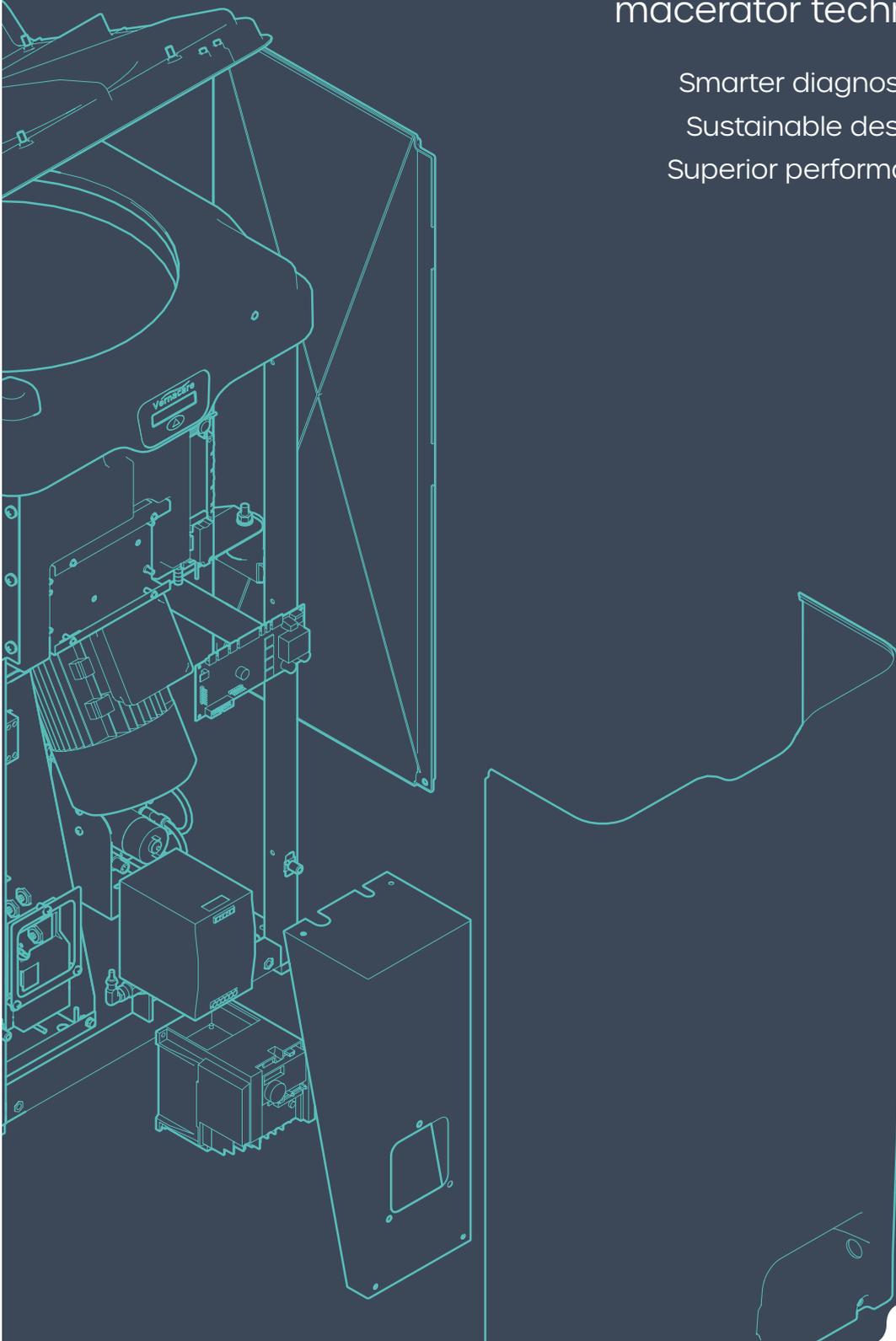

Vernacare

EBOS
HEALTHCARE

ESSENTIAL TO HEALTH

The new platform for macerator technology

The Vortex Air utilises the latest in motor control and diagnostics tools to minimise machine downtime whilst our SmartFlow™ maceration system ensures maximum performance. The smooth design and antimicrobial surfaces also work to reduce the risk of HCAs alongside our sensor controlled touchless technology operation.



Key Features & Benefits



Latest technology – including a deodorizer pump, capacitive sensors to measure the deodorizer and water tank levels



Smart sensors – for accurate and precise diagnostics ensuring optimal performance



Water efficient – processes four pulp items, saving on a valuable resource



Multi-cycle to suit all healthcare facilities – Eco (16L of water) Standard (23L of water) Heavy (27L of water)



Inflatable lid seal – prevents aerosolization of potentially harmful pathogens



Display – provides status of cycle during operation and time remaining to users



Multiple language options – available for international markets



Quiet operation – the quietest full-size macerator on the market at 62.0dBA

EBOS Code: VORTEXAIR

Working smarter to keep your teams safer

At Vernacare, we understand that machine downtime is not just disruptive, it can also pose an infection risk. The Vortex Air contains the most comprehensive diagnostics tools and sensors ever seen on a hospital macerator. Each unit is also tested and optimised to macerate our range of medical pulp products, ensuring your machine is running in even the most heavy-duty environments.

Why choose a single-use disposable system?



Eliminates cross infection risk from reusable bedpans



More convenient and hygienic



Zero aerosols released during disposal, eradicates risk of transmission via droplets




Hands-free system reduces number of potentially infected touch points



Immediate disposal reduces potential risk of contamination spray, splash backs, or spillages during cleaning



Vernacare macerators use significantly less water than bedpan washers. In addition to this, the Vortex Air only requires cold water to process pulp products resulting in less energy consumption per product.

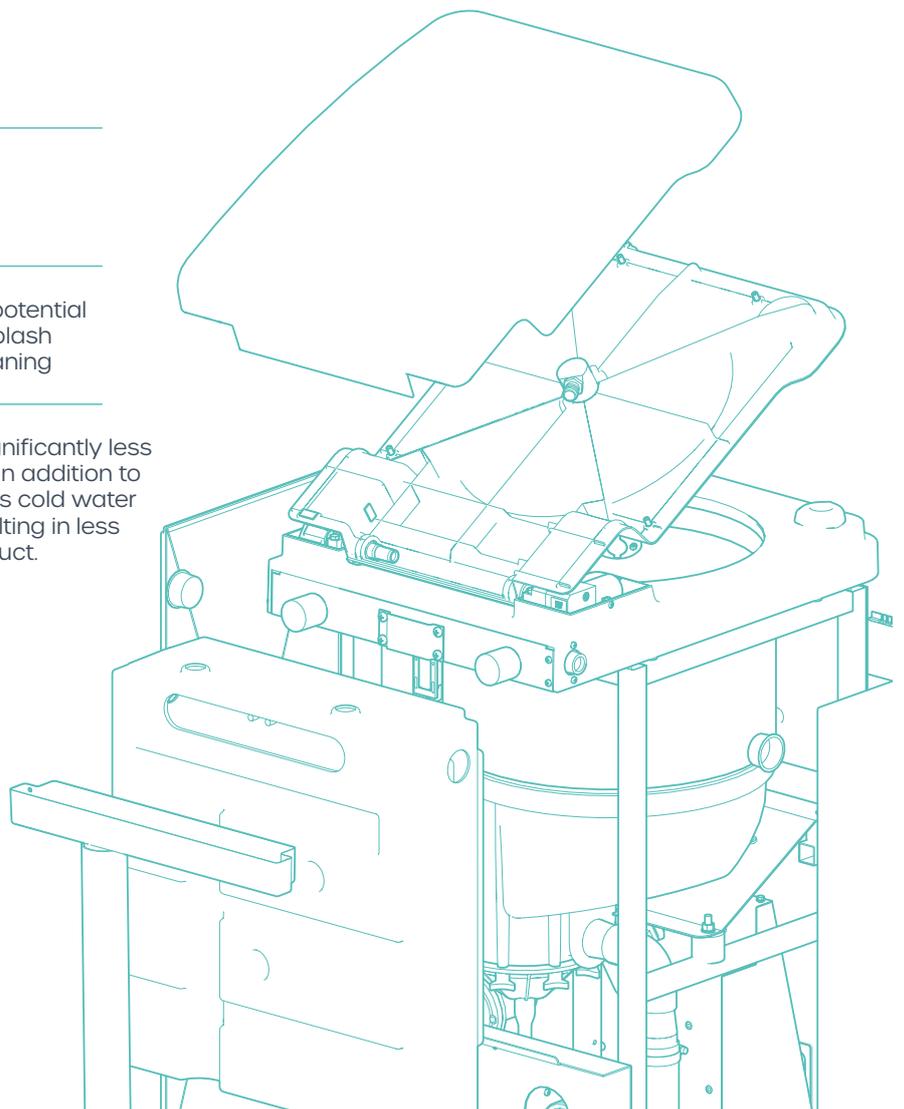

SMARTFLOW

TECHNOLOGY

Vernacare invented SmartFlow™ technology to protect the environment and the healthcare facility. Our unique 'closed-drum' system ensures that all waste placed into our machines is fully broken down before being released into the wastewater system.



Step 1
Contents Added



Step 2
Cycle Initiation



Step 3
Ultra-Fine Maceration



Step 4
Rapid Discharge

Upon discharge, when crushed and disintegrated by the macerator, the **Vernacare pulp** decomposes into particles, **95%** of which are less than **1mm** and are easily transported by the sewage pipes to the wastewater treatment plants (which is the same consistency as toilet paper). Studies have proven our discharge flow rate is 4x faster than any of our competitors.






Vernacare's SmartFlow™ Technology and pulp has been independently verified by EDANA.

Technical Specifications

The connection points and dimensions of the Vortex Air have been carefully chosen to allow seamless replacement of its predecessors in the range. This includes the V2000, V2020, Vortex and Vortex+.

EBOS Code: VORTEXAIR

Cycle Time (Standard)	120 seconds
Cycle Time (Heavy Duty)	140 seconds
Cycle Time (Eco Mode)	95 seconds
Weight	76kg
Capacity	6 small, 4 medium or 2 large disposable pulp products
Closing Mechanism	Automatic (via foot sensor)
Opening Mechanism	Automatic (via foot sensor)
Outside Dimensions	427 x 628 x 1027mm
Drain Outlet	2" P Trap (included)
Water Inlet	3/4" BSP (male)
Water Usage (Standard)	23L
Water Usage (Heavy Duty)	27L
Water Usage (Eco Mode)	16L
Required Flowrate	12L/min (minimum)
Sound Level	62 dBA (average)

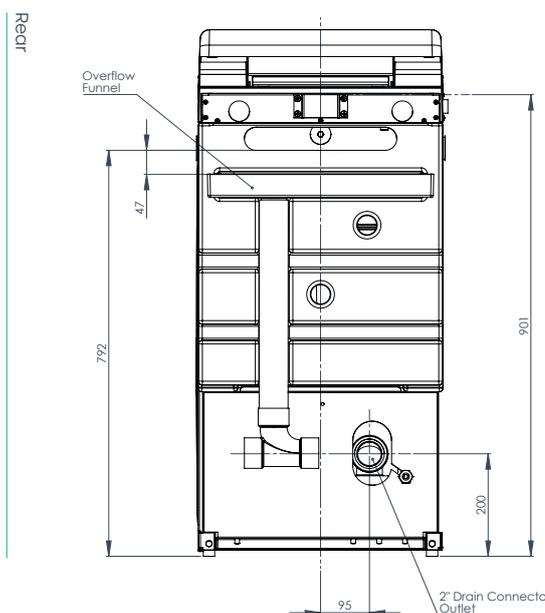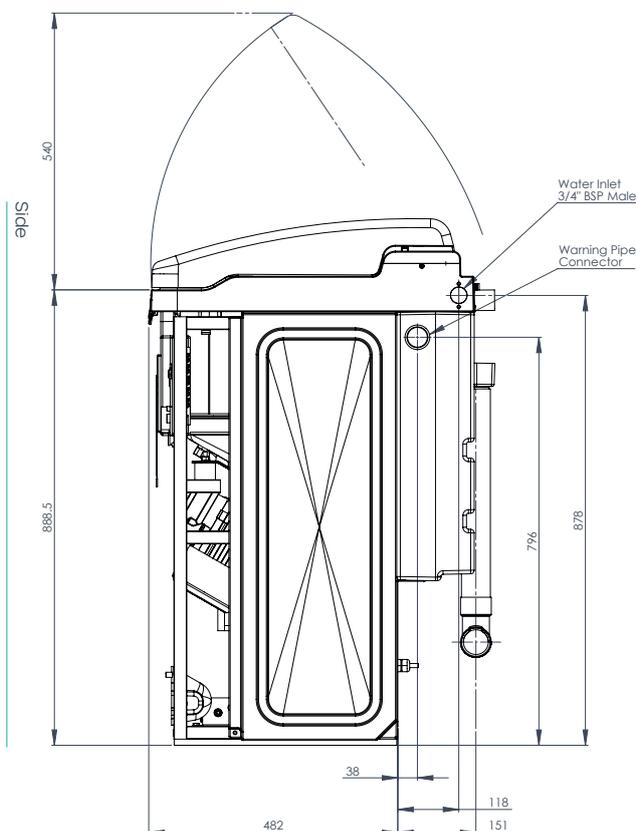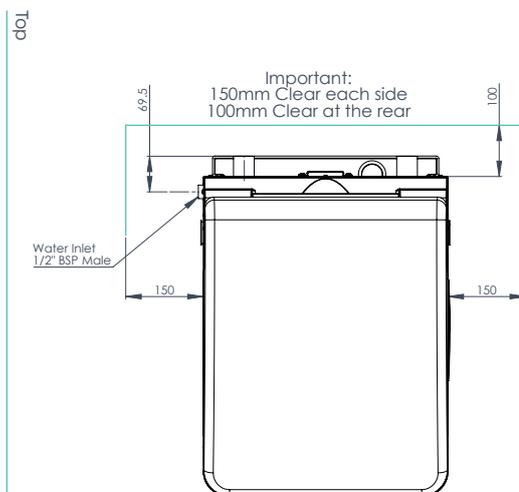

EBOS HEALTHCARE:

300 Victoria Street,
Wetherill Park,
NSW 2164

Free Phone: 1800 269 534

Free Fax: 1800 810 257

Email: customerservice@ebosgroup.com.au

Online Ordering: www.eboshealthcare.com.au

WAREHOUSE LOCATIONS:

New South Wales:

300 Victoria Street,
Wetherill Park,
NSW 2164

Queensland:

Unit 2, 39 Distribution Street
Motorway Business Park
Larapinta
QLD 4110

Victoria:

Unit 2A, 1508 Centre Road
Clayton Business Park
Clayton
VIC 3168

Western Australia:

2/3 Ernest Clark Road
Canning Vale
WA 6155

South Australia:

75-83 Hardys Road
Underdale
SA 5032

Tasmania:

7A Pear Avenue
Derwent Park
TAS 7009



ESSENTIAL TO HEALTH